



INTRODUCING

ADEXUSDx[®] hCG Rapid Blood Test

Featuring Touch-to-Test Technology

Sample is applied directly to the tip of the test

The ADEXUSDx[®] hCG Test is a rapid blood-based immunoassay that detects pregnancy earlier than urine-based tests. It provides in-field, lab accurate results within 10 minutes, using as little as a drop of blood.

SIMPLE


- **NO** buffer or external reagents needed
- **NO** lab equipment or readers
- **NO** waiting on urine samples

ACCURATE

- >99% sensitivity and specificity
- Fewer false negatives

BENEFITS


- Uses capillary or venous whole blood, plasma or serum
- Store at room temperature
- Self-contained and portable
- 10mIU/mL detection

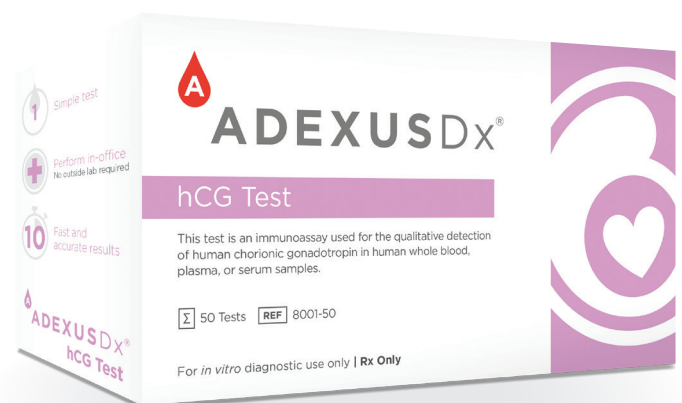

Administering Test*

FINGER STICK METHOD

1. Perform finger stick.

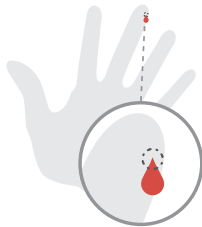

2. Touch the drop of blood to the tip of the test.




3. Set timer for 10 minutes and read result.



TRANSFER DEVICE METHOD

1. Collect 40 µL of sample using a liquid transfer device.

2. Apply sample to the test.

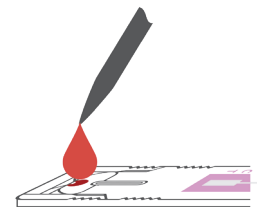


3. Set timer for 10 minutes and read result.



INTERPRETATION OF RESULTS

A positive result has TWO LINES, one at the “C” (Control Line) and one at the “T” (Test Line). The intensity of the lines may vary. Look carefully as **lines might be faint**.

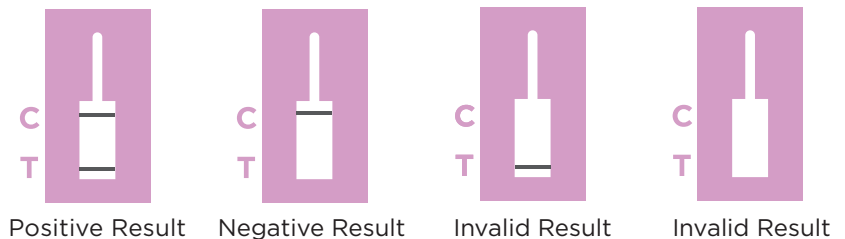

* For complete instructions, refer to the Product Insert before administering the test.

For more information email us at sales@nowdx.com.

Description

ADEXUSDx® hCG Test, Box
 ADEXUSDx® hCG Test, Box
 ADEXUSDx® hCG Test, Case

Quantity

25
 50
 1000

Catalog Number

8025-25
 8025-50
 8025-1000



1200 Stewart Place | Springdale AR 72764

phone 479.966.4530
 website nowdx.com