

# ADEXUSDx® hCG


## Rapid Blood Test & Control Set

Featuring Touch-to-Test  
Technology



### What is the ADEXUSDx® hCG Test?

The ADEXUSDx® hCG Test is a rapid blood-based immunoassay that detects pregnancy earlier than urine-based tests. It provides in-field, lab accurate results within 10 minutes, using as little as a drop of blood.

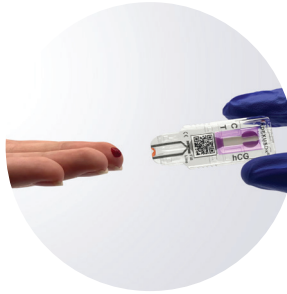

### What is the ADEXUSDx® hCG Control Set?

The ADEXUSDx® hCG Control Set is an unassayed quality control (QC) set designed for monitoring the reliability of tests for the qualitative detection of human chorionic gonadotropin.



### Simple

- **NO** external reagents needed
- **NO** lab equipment or readers
- **NO** waiting on urine samples

### Accurate

- >99% sensitivity and specificity
- Fewer false negatives


### Benefits

- Uses capillary or venous whole blood, plasma or serum
- Store at room temperature
- Self-contained and portable
- 10mIU/mL detection

# Administering the ADEXUSDx<sup>®</sup> hCG Test\*

FDA Cleared | CLIA Waiver Pending

## FINGER STICK METHOD

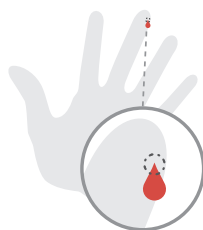


1. Perform finger stick.
2. Touch the drop of blood to the tip of the test.
3. Set timer for 10 minutes and read result.




## TRANSFER DEVICE METHOD

1. Collect 40  $\mu$ L of sample using a liquid transfer device.
2. Apply sample to the test.
3. Set timer for 10 minutes and read result.

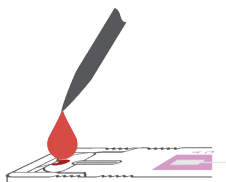

## INTERPRETATION OF RESULTS

A positive result has TWO LINES, one at the “C” (Control Line) and one at the “T” (Test Line). The intensity of the lines may vary. Look carefully as **lines might be faint**.

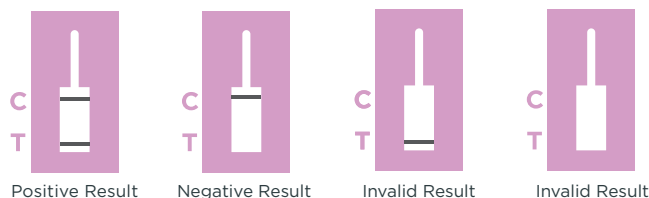

\* For complete instructions, refer to the Product Insert before administering the test.

For more information email us at [cc@nowdx.com](mailto:cc@nowdx.com)

### Description

ADEXUSDx<sup>®</sup> hCG Test, Box  
ADEXUSDx<sup>®</sup> hCG Test, Box  
ADEXUSDx<sup>®</sup> hCG Test, Case  
ADEXUSDx<sup>®</sup> hCG Control Set, Box

### Quantity

25  
50  
1000  
1

### Catalog Number

8001-25  
8001-50  
8001-1000  
401.009



NOWDiagnostics<sup>®</sup>

1200 Stewart Place | Springdale AR 72764

phone 479.966.4530  
website [nowdx.com](http://nowdx.com)