

# Syphilis Test & Control Set

## FEATURING

Revolutionary Lateral  
Flow Technology



## WHAT IS THE ADEXUSDx<sup>®</sup> Syphilis Test?

The ADEXUSDx<sup>®</sup> Syphilis Test is a rapid immunochromatographic assay used for the qualitative detection of the presence or absence of antibodies to syphilis in human whole blood, plasma, or serum to aid in the diagnosis of *Treponema pallidum*.



## WHAT IS THE ADEXUSDx<sup>®</sup> Syphilis Control Set?

The ADEXUSDx<sup>®</sup> Syphilis Control Set is an unassayed quality control (QC) set designed for monitoring the reliability of tests for the detection of antibodies to syphilis.



## Simple, Accurate & Low Cost

### Features & Benefits:

- 100% sensitive for any stage of syphilis
- 100% specificity
- Detected syphilis as early or earlier than 12 other tests in seroconversion panel

### Not Needed:

- Buffers or diluents
- In demand ancillary items
- Refrigeration


Learn more at [nowdx.com](http://nowdx.com)

phone 479.966.4530 | website [nowdx.com](http://nowdx.com)

# Administering the ADEXUSDx<sup>®</sup> Syphilis Test\*


1

Collect 40 µL of sample using a transfer device.

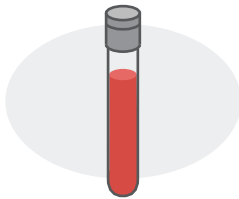

2

Apply sample to the test.

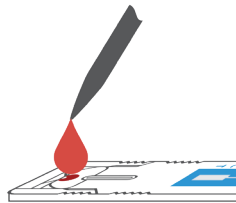


3

Set timer for 15 minutes and read result.



## Interpretation of Results

A positive result has TWO LINES, one at the “C” (Control Line) and one at the “T” (Test Line). The intensity of the lines may vary. Look carefully as **lines might be faint.**

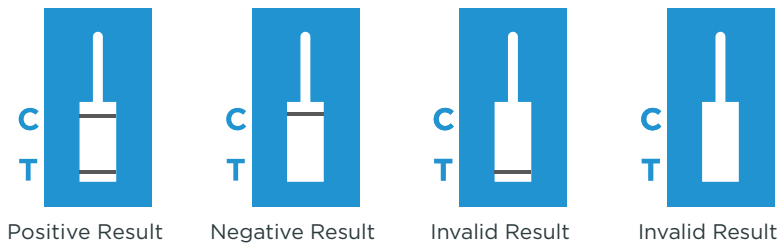

Positive Result

Negative Result

Invalid Result

Invalid Result

\* For complete instructions, refer to the Package Insert before administering the test.

Visit [nowdx.com](http://nowdx.com) for more information.

Description	Quantity	Catalog Number
ADEXUSDx <sup>®</sup> Syphilis Test, Box	25	8057-25
ADEXUSDx <sup>®</sup> Syphilis Test, Box	50	8057-50
ADEXUSDx <sup>®</sup> Syphilis Test, Boxes of 25, Case	1000	8057-1000-25
ADEXUSDx <sup>®</sup> Syphilis Test, Boxes of 50, Case	1000	8057-1000-50
ADEXUSDx <sup>®</sup> Syphilis Control Set, Box	1	457.006

NOW Diagnostics<sup>®</sup>

1200 Stewart Place | Springdale AR 72764  
phone 479.966.4530 | website [nowdx.com](http://nowdx.com)

