


ADEXUSDx® COVID-19

Total Antibody Rapid Test & Control Set

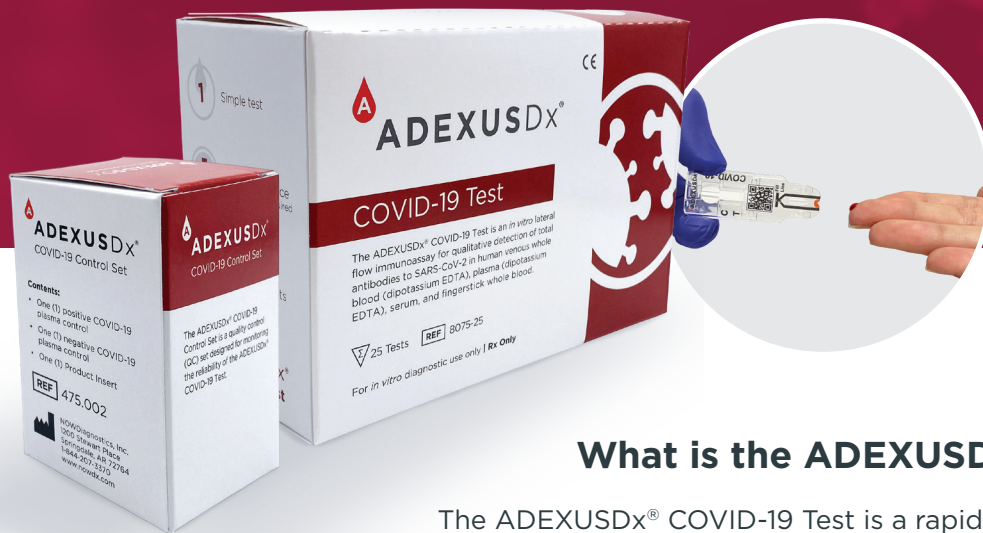


Featuring
Touch-to-Test
Technology

What is the ADEXUSDx® COVID-19 Test?

The ADEXUSDx® COVID-19 Test is a rapid serology test that detects total antibodies to SARS-CoV-2. It provides in-field, lab accurate results within 15 minutes, using as little as a drop of blood.

What is the ADEXUSDx® COVID-19 Control Set?

The ADEXUSDx® COVID-19 Control Set is a quality control (QC) set designed for monitoring the reliability of the ADEXUSDx® COVID-19 Test.

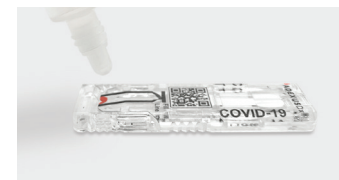

Simple, Accurate & Low Cost


- **NO** buffers or external reagents needed
- **NO** lab equipment
- **NO** in demand ancillary items
- **NO** refrigeration
- Highly sensitive and specific
- Detects antibodies to the RBD section of the spike protein
- Less than a co-pay

Administering the ADEXUSDx[®] COVID-19 Test*

For use under Emergency Use Authorization only


LABORATORY PIPETTE METHOD

1. Collect 40 µL of sample using a laboratory pipette device.
2. Apply sample to the test.
3. Set timer for 15 minutes and read result.

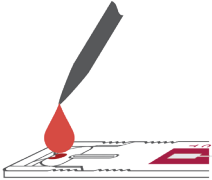


FINGER STICK METHOD

1. Perform finger stick.
2. Touch the drop of blood to the tip of the test.
3. Set timer for 15 minutes and read result.

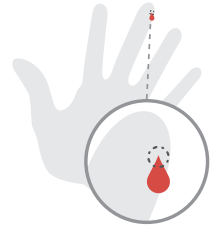

INTERPRETATION OF RESULTS

A positive result has TWO LINES, one at the “C” (Control Line) and one at the “T” (Test Line). The intensity of the lines may vary. Look carefully as **lines might be faint**.

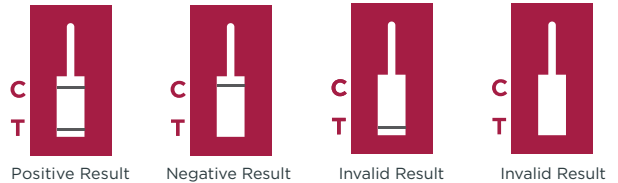

* For complete instructions, refer to the Product Insert before administering the test.

How to perform QC with the ADEXUSDx[®] COVID-19 Control Set

1. Follow the COVID-19 Control Set Instructions For Use included with the controls
2. Scan this QR Code to watch a step-by-step video



Visit [C19Development.com](https://www.C19Development.com) for more information.

Description	Quantity	Catalog Number
ADEXUSDx [®] COVID-19 Test, Box	25	8075-25
ADEXUSDx [®] COVID-19 Test w. Lancets, Box	25	8075-25-FS
ADEXUSDx [®] COVID-19 Test, Box	50	8075-50
ADEXUSDx [®] COVID-19 Test w. Lancets, Box	50	8075-50-FS
ADEXUSDx [®] COVID-19 Test, Case	1000	8075-1000
ADEXUSDx [®] COVID-19 Test w. Lancets, Case	1000	8075-1000-FS
ADEXUSDx [®] COVID-19 Control Set, Box	1	475.002



Testing of venous whole blood (dipotassium EDTA), plasma (dipotassium EDTA), and serum specimens is limited to laboratories certified under the Clinical Laboratory Improvement Amendments of 1988 (CLIA), 42 U.S.C 263a, that meet requirements to perform moderate or high complexity tests. Testing of fingerstick whole blood specimens is limited to laboratories certified under the Clinical Laboratory Improvement Amendments of 1988 (CLIA), 42 U.S.C. §263a, that meet the requirements to perform high, moderate, or waived complexity tests. Testing of fingerstick whole blood specimens is authorized for use at the Point of Care (POC), i.e., in patient care settings operating under a CLIA Certificate of Waiver, Certificate of Compliance, or Certificate of Accreditation.